



Model: Flex 5000, TR 5000, RR Flex

Putting the system into operation | In Betrieb nehmen |
Mise en service du système | Puesta en servicio | Messa
in funzione | In gebruik nemen | Colocar em
funcionamento | Uruchamianie

Power

Model: TR 5000

Model: NT5-10AW

EU

UK

US


AU

Publ. 09/22, 570737/A04

2


 

[www.sennheiser-hearing.com/
how-to-videos](http://www.sennheiser-hearing.com/how-to-videos)

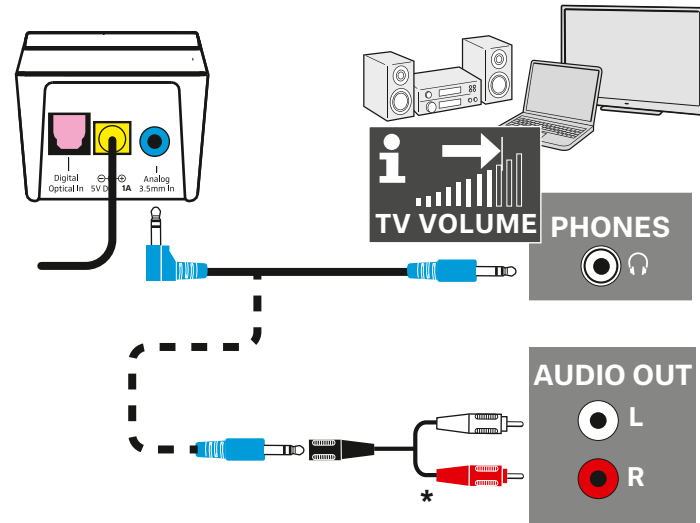



The diagram illustrates the connection of an optical cable. On the left, a source device (like a Blu-ray player or game console) has three output ports: Digital Optical In (red), S/PDIF (yellow), and Analog 3.5mm In (blue). A red optical cable is plugged into the Digital Optical In port. On the right, a display device (like a TV or monitor) has an 'OPTICAL OUT' port (red). The other end of the red optical cable is plugged into this port. The background shows a living room setting with a TV, speakers, and a game console.

Digital - no sound ?

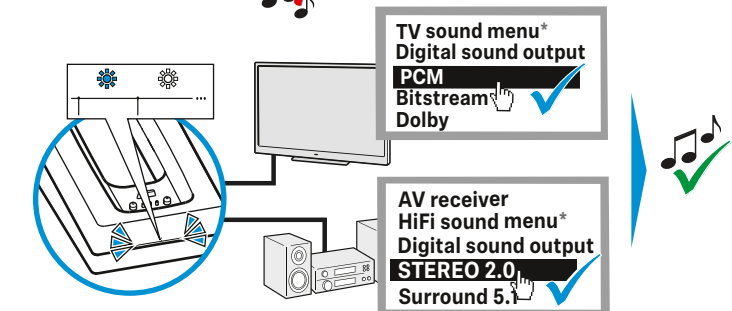

*Sound menu may vary

Charging the receiver's rechargeable battery |
Empfängerakku laden | Charger l'accu du récepteur |
Cargar la pila recargable del receptor | Caricamento della
batteria ricevitore | De accu van de ontvanger opladen |
Carregar a bateria do recetor | Ładowanie akumulatora
odbiornika

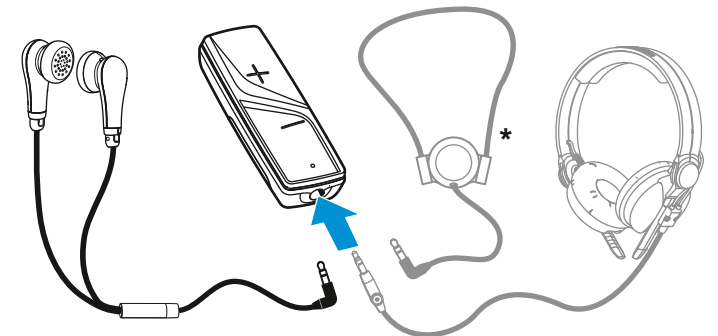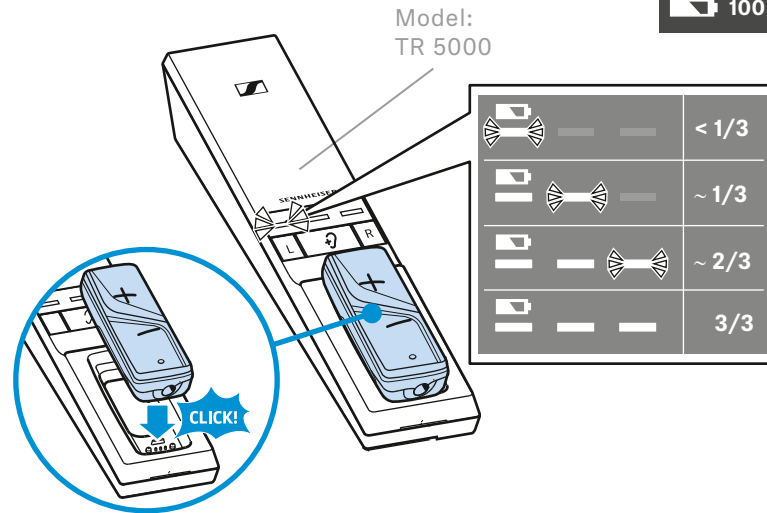

*** to be ordered separately | Nicht im Lieferumfang enthalten | à commander séparément | No se incluye en el volumen de suministro | Non in dotazione | Wordt niet meegeleverd | Não incluído | Nie należy do zestawu**





www.sennheiser-hearing.com/download


EN NL NO JA LT BG
DE PT SV RU SK SL
FR PL CS ET HU HR
ES DA EL LV RO
IT FI TR



High Volume

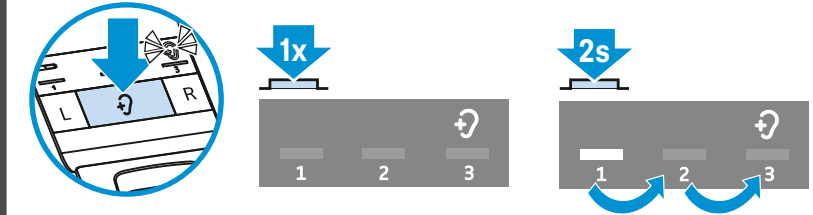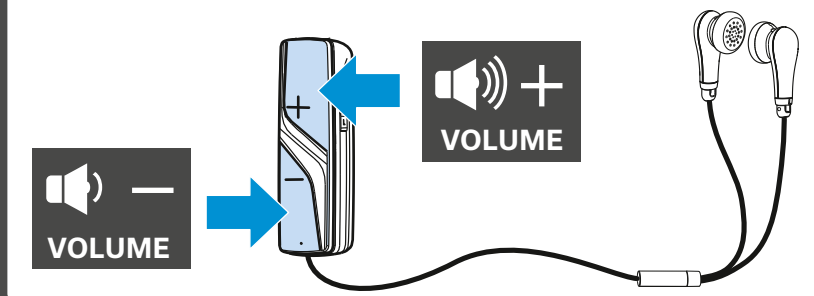

L	 	±
±	 	R

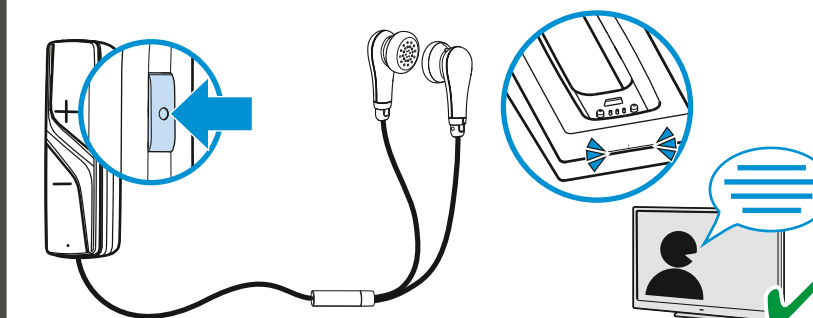



SENNHEISER

Flex 5000


Digital Wireless audio system for headphones

Model: Flex 5000, TR 5000, RR Flex

Kort vejledning | Pikaohje | Kort veiledning |
Snabbguide | Rychlý průvodce | Σύντομες οδηγίες |
Kısa Kılavuz | クイックガイド | Краткая инструкция

Ibrugtagning | Käyttöönotto | Sette i drift | Installera
| Uvedení do provozu | Έναρξη λειτουργίας |
Devreye alma | セットアップする | Ввод в работу

1 Tilslutning af sender til strømnettet | Lähettimen
yhdistäminen sähköverkkoon | Koble sender til strømnettet |
Anslut sändaren till elnätet | Pripojení vysílače k elektrické
síti | Σύνδεση πομπού στο ηλεκτρικό ρεύμα | Vericinin
elektrik şebekesine bağlanması | Трансмисmitterを電源に接続
する | Присоединение передатчика к электрической сети

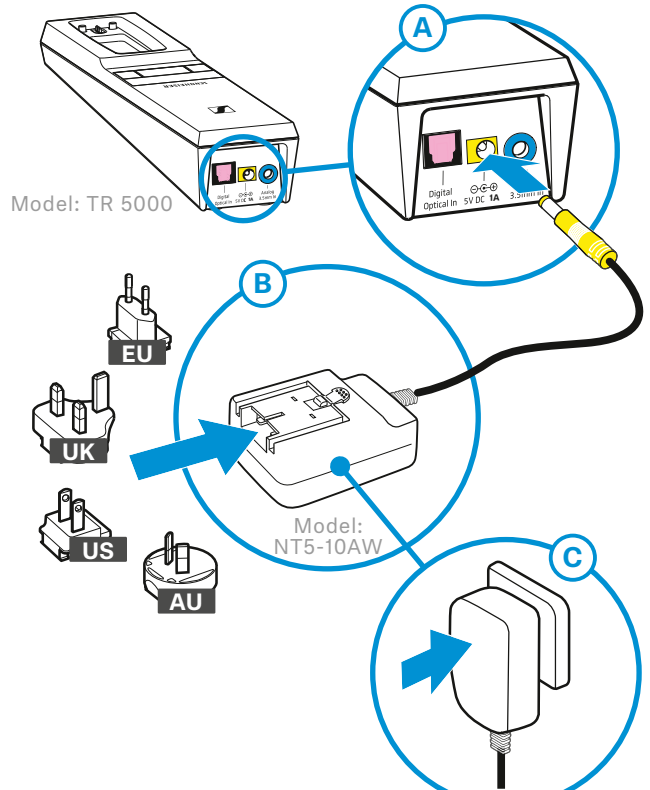

Sonova Consumer Hearing GmbH
Am Labor 1, 30900 Wedemark, Germany
www.sennheiser-hearing.com


Publ. 09/22, 570737/A04

2


Tilslutning af sender til en
audiokilde | Lähettimen
yhdistäminen äänilähteeseen |
Koble sender til lydkilde | Anslut
sändaren till en ljudkälla |
Pripojení vysílače ke zdroji zvuku
| Σύνδεση πομπού σε μια πηγή ήχου | Vericinin bir ses kaynağına
bağlanması | Трансмисmitterをオーディオソースに接続する |
Присоединение передатчика к источнику аудиосигнала




www.sennheiser-hearing.com/
how-to-videos

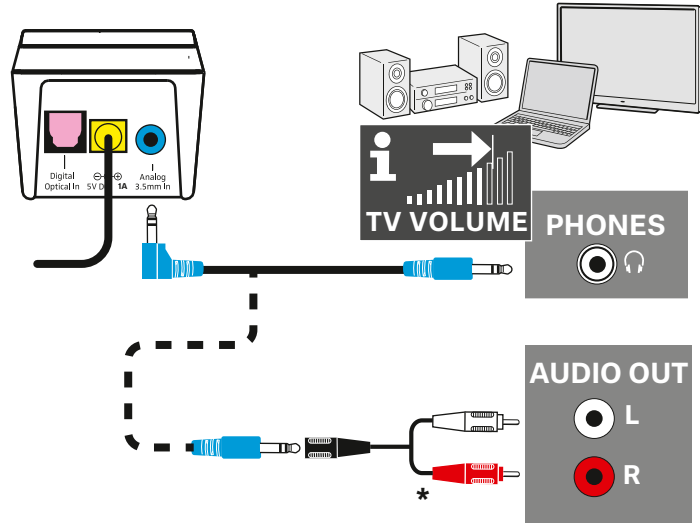

* Ikke indeholdt i leveringsomfanget | Ei sisälly toimitukseen | Ikke
vedlagt leveransen | Ingår inte | Není součástí dodávky | Δεν
περιλαμβάνονται στη συσκευασία παράδοσης | Teslimat kapsamına
dahil değildir | 同梱されていません | Не входит в комплект поставки

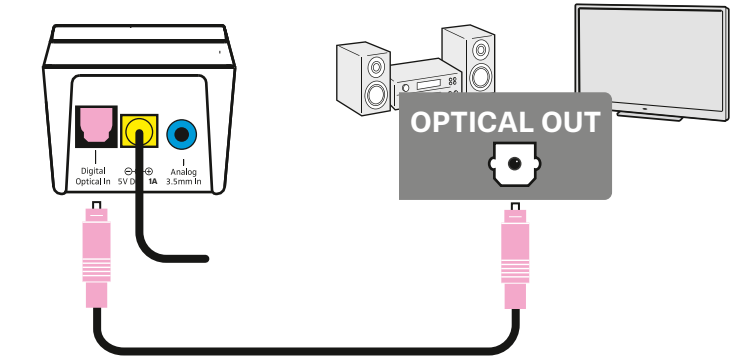


Digital - no sound ?

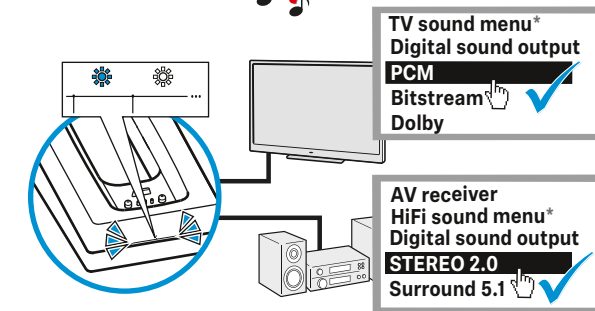


*Sound menu may vary

Anvendelse | Käyttö | Bruke | Användning | Použití | Χρήση
| Kullanma | 使用する | Использование

Opladning af modtagerbatteri | Vastaanottimen akun
lataaminen | Lade mottakerbatteri | Ladda mottagarens
batteri | Nabiti baterie přijímače | Φόρτιση μπαταρίας
δέκτη | Alıcı aküsünü şarj etme | 受信機のバッテリーを充電
する | Зарядка аккумулятора приемника

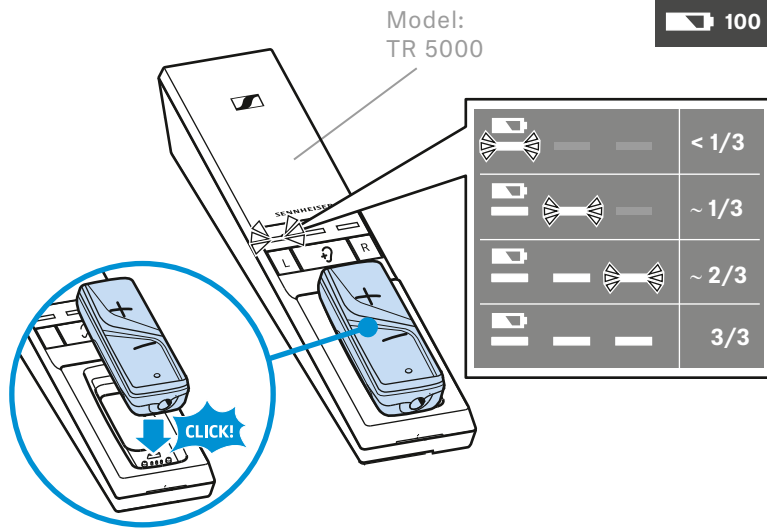

Tilslutning af hovedtelefon eller teleslynge |
Kuulokkeiden tai induktiosilmukan yhdistäminen | Koble
til hodetelefon eller induksjonsslynge | Anslut hörlurarna
eller induktionsslingan | Pripojení sluchátek nebo
indukční smyčky | Σύνδεση ακουστικών ή επαγωγικού
βρόχου | Kulaklığı ya da endüksiyon halkasını bağlama |
ヘッドホンまたはインダクションネックループを接続する |
Подключение наушников или индукционной петли

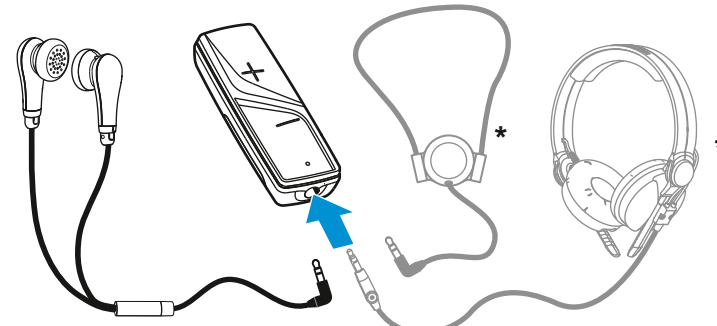


* Ikke indeholdt i leveringsomfanget | Ei sisälly toimitukseen | Ikke vedlagt
leveransen | Ingår inte | Není součástí dodávky | Δεν περιλαμβάνονται
στη συσκευασία παράδοσης | Teslimat kapsamına dahil değildir |
同梱されていません | Не входит в комплект поставки




www.sennheiser-hearing.com/download

EN NL NO JA LT BG
DE PT SV RU SK SL
FR PL CS ET HU HR
ES DA EL LV RO
IT FI TR


Tænd/sluk | Virran kytkeminen ja katkaiseminen | Slå av/på | Sätt
på/stäng av | Vyrnuti/zapnuti | Ενεργοποίηση/απενεργοποίηση |
Аçma/Kapatma | オン/オフにする | Включение/выключение

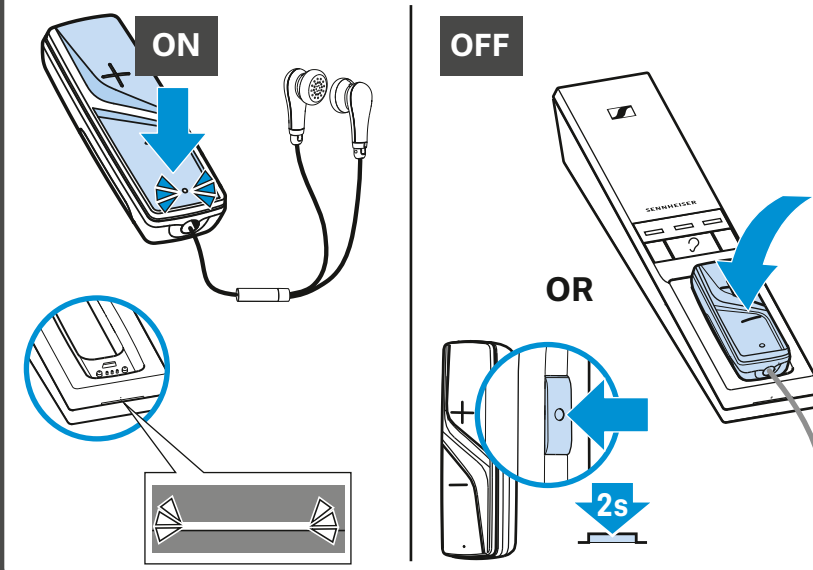


Indstilling af lydstyrke | Äänenvoimakkuuden säätäminen |
Stille inn lydstyrke | Ställ in volymen | Nastavení hlasitosti |
Ρύθμιση έντασης | Ses düzeyini ayarlama | 音量を調節する |
Регулировка громкости звука

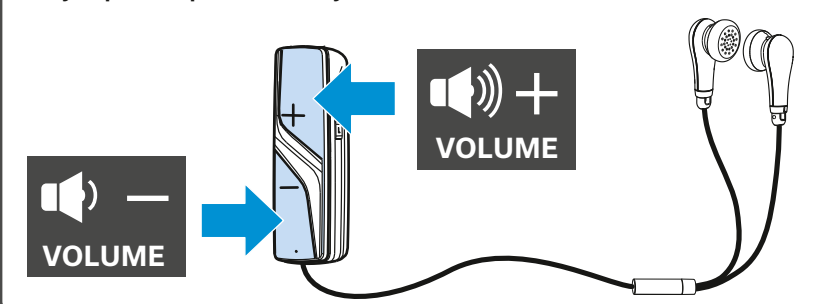

Fastgørelse af modtager på tøj | Vastaanottimen kiinnittäminen
vaatetukseen | Fest mottaker på klesplagg | Fäst mottagaren på
kläderna | Pripěvnění přijímače na oděv | Στερέωση του δέκτη στο
ρούχο | Alıcıyı giysi üzerinde sabitleme | 受信機を服に固定する |
Крепление приемника на одежде

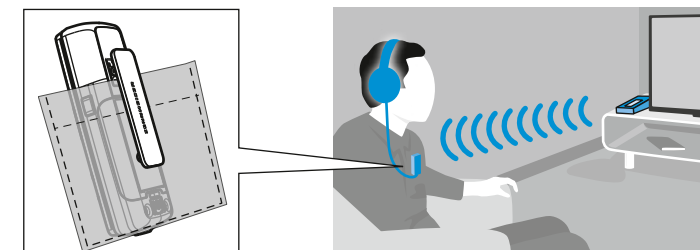

Valg af profil til lydforbedring | Äänenparannustoiminnon
profiilin valitseminen | Velg profil for forbedret lyd | Välj
profil för bättre klang | Vyběr profilu pro zlepšení zvuku |
Επιλογή προφίλ για τη βελτίωση του ήχου | Ses düzeltme
profilini seçme | サウンド改善機能のためのプロファイルを選択
する | Выбор профиля оптимизации звучания

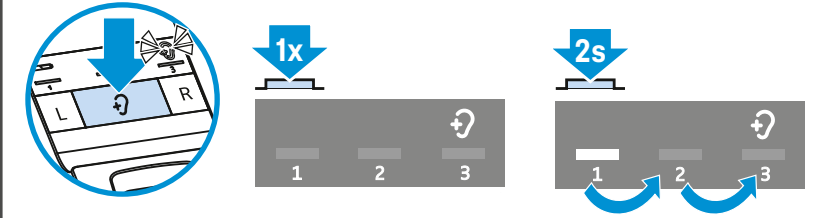


Indstilling af balance | Kanavatasapainon
säätäminen | Innstill likevekten | Ställ in balans |
Nastavení vyvážení zvuku | Ρύθμιση ισορροπίας |
Balansı ayarlama | バランスを調整する | Настройка
баланса



Nulstilling af balance | Kanavatasapainon
palauttaminen lähtötilaan | Tillbakeställ likevekten |
Återställ balans | Reset vyvážení zvuku |
Επαναφορά ισορροπίας | Balansı sıfırlama |
Баланс을 리셋하는 | Сброс баланса



Tilkobling af taleforståelighed | Puheenselkeytystoiminnon
kytkeminen | Slå på språkforståelse | Aktivera
taförståelse | Zapnutí srozumitelnosti řeči |
Ενεργοποίηση καθαρότητας ομιλίας | Konuşma
anlaşılabilirliğini açma | 音声了解度をオンにする | Включение
режима понятности речи

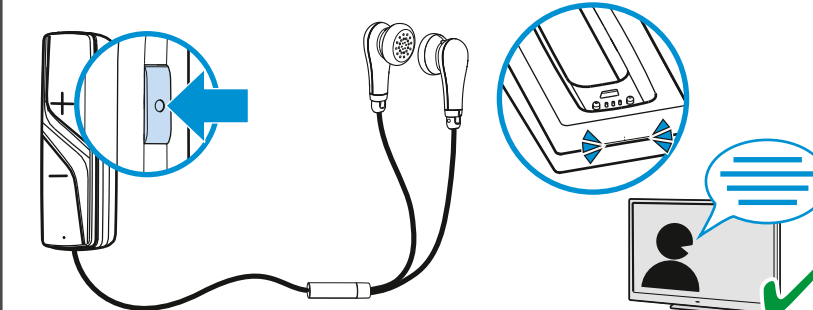



SENNHEISER

Flex 5000


Digital Wireless audio system for headphones

Model: Flex 5000, TR 5000, RR Flex

Lūhijuhend | Īsā instrukcija | Trumpoји naudojimo instrukcija | Struný návod | Rövid útmutató | Instrucțiuni succinte | Кратко ръководство | Kratka navodila | Kratke upute

Kasutuselevõtt | Eksploatačijas sākšana | Paruošti naudoti | Uvedenie do prevádzky | Üzembe helyezés | Punerea în funcțiune | Въвеждане в експлоатация | Zagon | Pustiti u rad

1 Saatja ühendamine vooluvõrku | Raiditāja pieslēgšana pie elektrotīkla | Siųstuvą prijungti prie elektros tinklo | Pripojenie vysieláča do siete | Az adó csatlakoztatása a villamos hálózathoz | Conectarea emițătorului la rețeaua de curent electric | Свързване на предавателя към електрическата мрежа | Priključitev oddajnika na električno omrežje | Priključiti odašiljač na strujnu mrežu

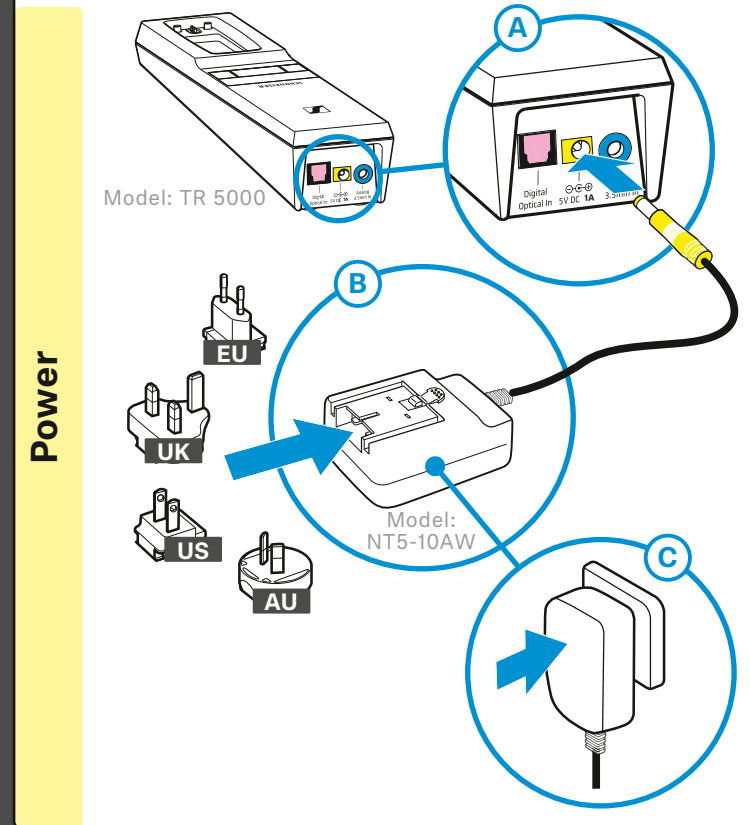


Sonova Consumer Hearing GmbH
Am Labor 1, 30900 Wedemark, Germany
www.sennheiser-hearing.com


Publ. 09/22, 570737/A04

2


Saatja ühendamine audioallikaga | Raiditāja pieslēgšana pie audio avota | Siųstuvą prijungti prie garso šaltinio | Pripojenie vysieláča k zdroji zvuku | Az adó csatlakoztatása egy hangforráshoz | Conectarea emițătorului la sursa audio | Свързване на предавателя към аудио източник | Priključitev oddajnika na vir zvoka | Priključiti odašiljač na izvor zvuka

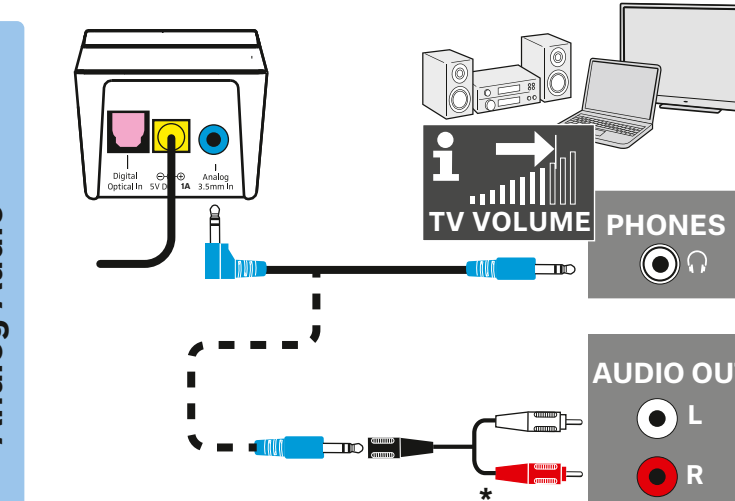

* Ei sisaldu tarnekomplektis | Nav iekļauts komplektācijā | Nejeina i tiekimo paketo sudėtį | Nie je súčasťou balenia | Nincs benne a kiszállított csomagban | Neinclus(ă) în pachetul de livrare | He e включено в обема на доставката | Ni v obsegu dobave | Nije sadržano u opsegu isporuke

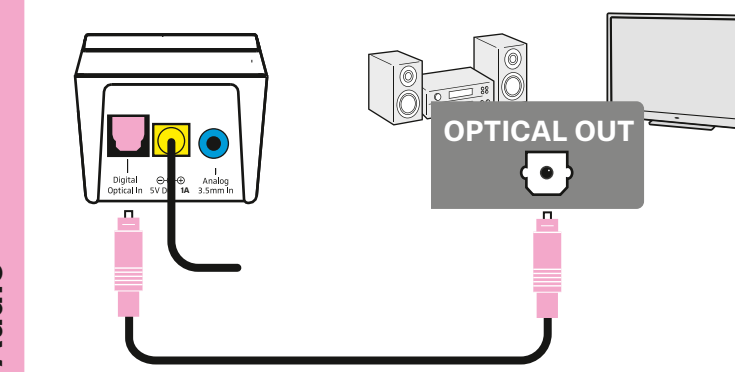

Digital - no sound ?

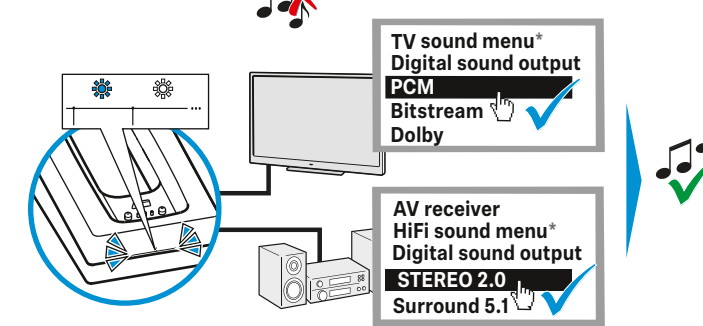


Analog Audio

Digital Audio


*Sound menu may vary

Kasutamine | Izmantošana | Naudoti | Používanie | Alkalma-
zás | Modul de utilizare | Употреба | Uporaba | Upotrijebiti

Lõuaaluse kinnitusega kõrvaklappide laadimine | Stetoskopa lādēšana | Įkrauti po smakru dėvimas ausines | Nabíjanie slúchadiel pod bradu | Az állpántos fejhallgató töltése | Încărcarea receptorului stetoscopic | Зарядете слушалките | Polnjenje podbradnih slušalk | Napuniti slušalice sa stremenom ispod brade

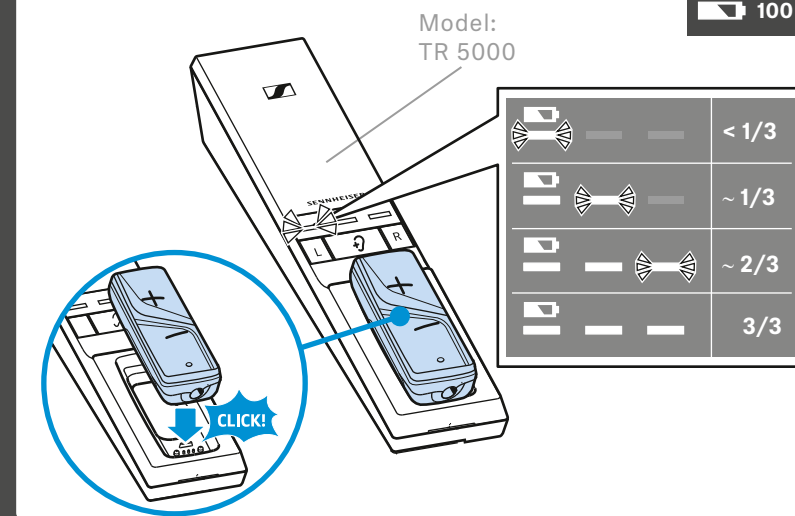


Kõrvaklappide või kuuldevõimendi ühendamine | Austiņu vai indukcijas cilpas pieslēgšana | Prijungti ausines arba indukcin kilp | Pripojenie slúchadiel alebo induknej sluky | A fejhallgató vagy az indukciós hurok csatlakoztatása | Conectarea c știlor sau a buclei de inducție | Свържете слушалки или индукционна намотка | Priključitev slušalk ali induktivne zanke | Priklju iti slušalice ili indukcijsku petlju

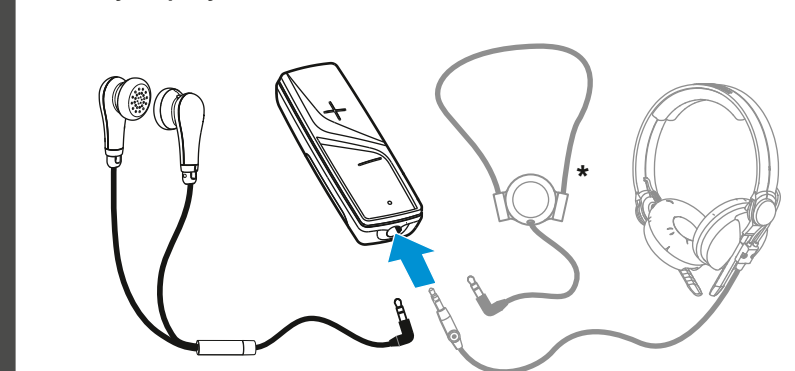


* Ei sisaldu tarnekomplektis | Nav iekļauts komplektācijā | Nejeina i tiekimo paketo sudėtį | Nie je súčasťou balenia | Nincs benne a kiszállított csomagban | Neinclus în pachetul de livrare | He e включено в обема на доставката | Ni v obsegu dobave | Nije sadržano u opsegu isporuke

www.sennheiser-hearing.com/download


Sisse-/väljalülitamine | Ieslēgšana/izslēgšana | Įjungti arba |
Zapnutie/vypnutie | Be-/kikapcsolás | Pornirea/oprirea |
Вкл./изключване | Vklp/izklop | Uključiti/isključiti

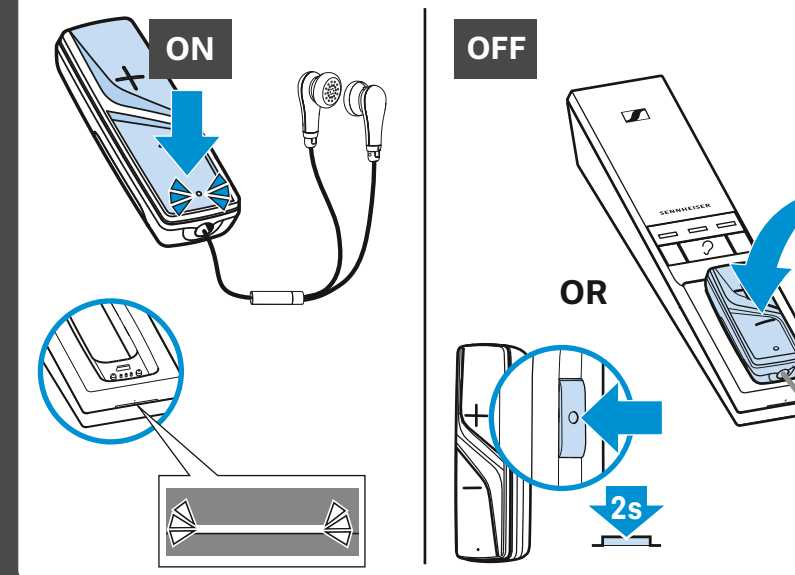


Helitugevuse seadmine | Skāļuma iestatīšana | Nustatyti garsumą | Nastavenie hlasitosti | A hangerő beállítása |
Reglarea volumului | Настройка силата на звука |
Nastavitev glasnosti | Podesiti glasnoću zvuka

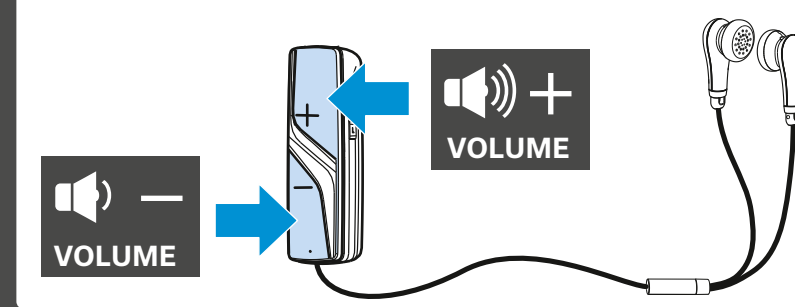

Vastuvõtja kinnitamine riiete külge | Uztvērēja piestiprināšana pie apģērba | Imtuvą pritvirtinti prie drabužių | Upevnenie prijamača na oblečenje | A vevőegység rögzítése a ruházaton | Fixarea emițătorului pe îmbrăcăminte | Закрепете приемника за дрехите | Pritrditev sprejemnika na oblačilo | Priključiti prijammik na odjeću

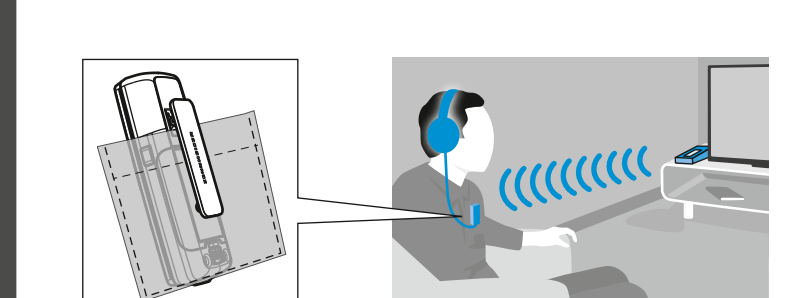

Heliparanduseks profiili valimine | Profila izvēle skaņas uzlabošanai | Pasirinkti profilį garsui pagerinti | Výber profilu na zlepšenie zvuku | A hangjavítási profil kiválasztása | Selectarea unui profil de amplificare a sunetului | Изберете профил за подобрене на звука | Izbira profila za izboljšanje zvoka | Odabratí profil za poboljšanje zvuka

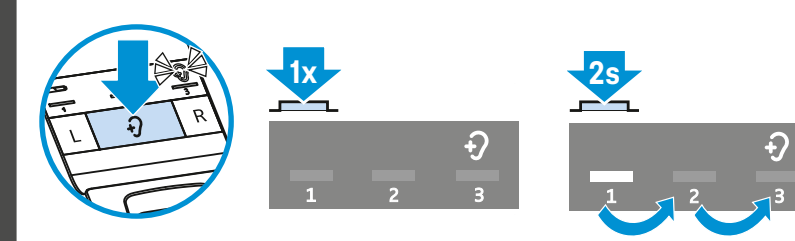

Helitasakaalu seadmine | Balansa regulēšana | Nustatyti garso balansą | Nastavenie rovnováhy | A balansz beállítása |
Reglarea balansului | Регулиране на баланса |
Nastavitev izravnave | Podesiti ravnotežu



Helitasakaalu lähtestamine | Balansa atiestatīšana | Atkurti garso balansą | Vynulovanie rovnováhy | A balansz visszaállítása |
Resetarea balansului | Нулиране на баланса |
Ponastavitev izravnave | Vratiti ravnotežu



Kõne mõistetakse sisselülitamine | Runas skaidrības ieslēgšana | Įjungti kalbos suprantamumo funkciją | Zapnutie zrozumiteľnosti jazyka | A beszéd érthetőségének beállítása | Pornirea funcției de inteligibilitate a limbajului | Включете говора | Vklp razumljivosti govora | Uključiti razumijevanje govorenog jezika



ET Eesti Keel | LV Latviešu Valoda | LT Lietuvių Kalba | SK Slovenščina | HU Magyar | RO Română | BG Български език | SL Slovenščina | HR Hrvatski Jezik